



Handheld X-ray Systems

News & User Report

Ensure maximum battery life for your NOMAD Pro!

Aribex recommends the following:

- ***Rotate your battery handsets routinely***
 - It is highly recommended that you rotate your battery handsets between the charger cradle and NOMAD Pro on a regular basis.
 - Our battery handsets prefer more exercise and use.
 - Discharging the battery handsets to zero is not required, though it may actually prolong their life.
 - Routinely using both handsets will also extend battery handset life.
- ***Do NOT store your NOMAD Pro for longer than 90 days without charging both battery handsets***
 - This is very important for newly purchased NOMADs that are stored while construction work is completed, etc.
 - This is also important for NOMAD Pro devices used only for periodic humanitarian trips.
 - Charge handsets anytime you expect the NOMAD Pro to be out of use over 90 days.
- ***How do you know the batteries are at the end of their life cycle?***
 - At the end of the life of the battery, the number of exposures per charge will decrease.

For any questions you may have regarding your NOMAD Pro please contact Aribex directly through our Orem, UT office at 1-801-226-5522 or via our website: www.aribex.com

Thank you for being a loyal NOMAD customer!



MC-0039 Rev. A



744 South 400 East
Orem, UT 84097
801-226-5522
www.aribex.com